

Red blood cell abnormalities

as seen with May-Grünwald-Giemsa (MGG) staining



Plasmodium falciparum inclusions



Description Ring type: brown nucleole, ringshaped blue cytoplasm, infested red blood cell not enlarged, no granulation or pigmentation, multiple parasites common

Incidence

Malaria tropica, >5% parasite infestation = severe malaria tropica. Plasmodium falciparum must be detected in the blood smear!

Erythroblast

Description

per 100 WBC.

Incidence

Erythroblasts (E) in peripheral

logical (physiological only in

blood of adults are always patho-

first month of life). In the blood

smear, E are not included in WBC

differential but counted on top

Associated with diseases show-

ing increased erythro- and hae-

matopoiesis and in extramedul-

lary haematopoiesis

Rouleaux



Description Linear arrangement of red blood cells simulating a 'stack of coins'

Incidence

Hypergammaglobulinaemia, monoclonal and polyclonal, hyperfibrinogenaemia

Normocytic red blood cell



Description Diameter: 7 µm, equals the diameter of the nucleus of a small lymphocyte. Colour: haemoglobin colour (pink – red); shape: round or oval, central pallor 1/3 of the cell surface; no inclusions.

Incidence Physiological red blood cell

Abbreviations: Ø – diameter, RNA – ribonucleic acid, DNA – deoxyribonucleic acid, MDS – myelodysplastic syndrome, AML – acute myeloid leukaemia, BM – bone marrow, Hb – haemoglobin, WBC – white blood cells

© Copyright 2015 – Universitätsklinikum Aachen AÖR, Klinik für Hämatologie, Onkologie, Hämostaseologie und Stammzelltransplantation Authors: Reinhild Herwartz, Prof. Dr. med. Roland Fuchs, Medizinische Klinik IV, Uniklinik RWTH Aachen

Sysmex Europe GmbH

Bornbarch 1, 22848 Norderstedt, Germany · Phone +49 40 52726-0 · Fax +49 40 52726-100 · info@sysmex-europe.com · www.sysmex-europe.com

In cooperation with:

Universitätsklinikum Aachen Klinik für Hämatologie, Onkologie, Hämostaseologie und Stammzelltransplantation Pauwelsstraße 30, 52074 Aachen, Germany · Phone +49 241 80-0 · info@ukaachen.de · www.ukaachen.de

You will find your local Sysmex representative's address under www.sysmex-europe.com/contacts

